

**SEILER**

# **IMAGE CAPTURE**


# *Pro*


**PRODUCT MANUAL**  
(COLPOSCOPES, ENT, DENTAL SCOPES)



The Seiler Image Capture Pro is a hardware and software package that allows the user to capture still images and record live streaming video directly from the CCD color camera. The Seiler Image Capture Pro has been specifically designed to easily capture and store images and live streaming video by the patients name and or Patient Number. These images will be stored within the hard drive of the computer that is provided with the Seiler Image Capture Pro Package. The images and videos can then be emailed, stored, transferred, or imported into a patient management system.

## INSTALLATION INSTRUCTIONS

- 1) Insert Capture Card into computer.
- 2) Connect the Capture Card to the S-Video on the High Resolution Video Camera, which is attached to the microscope, with cord provided.
- 3) Double click on  icon on the desktop.

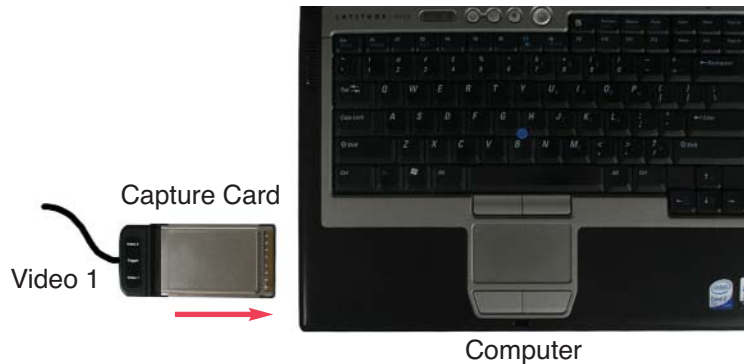

## SAVING A FILE

- 1) Click the browse button to identify the file location for images and video to be stored.



## EXITING THE PROGRAM

- 1) Click the ESC button.



For additional help setting up your Seiler Image Capture Pro system, please contact Seiler Precision Microscopes.



(800) 489-2282

(314) 968-2282

E-mail: [micro@seilerinst.com](mailto:micro@seilerinst.com)

[www.seilerinst.com](http://www.seilerinst.com)



Click the "Take Photo" button to capture the image to the designated file folder.

Click to start and stop recording video to the designated file folder.

Options button.

Print button.

On the bottom left hand of the screen you will see the **TAKE PHOTO** function. By clicking onto this button, the image that you are viewing will be captured as a still image and stored into the specified file under the specified name and/or patient number.

The next button to the right is the Start Video Function. By clicking on the **START VIDEO** button, you will start recording live streaming video to a specified file under the patients name or number. To stop recording you click onto the same button again and this is the **STOP VIDEO** Function.

The next button is the **OPTIONS** button. By clicking onto this button it will open the OPTIONS screen. In the options screen you will be able to change video settings, identify whether the images that are captured are saved as .bmp or .jpeg, select a codec for your video, and text overlay the time, date, and text. The system is currently set to the optimal settings to record and capture images. It is not recommended to change the video or movie options unless you are

technologically experienced with computers and computer formats.

The two white bars that you see are the User ID bar and the file destination bar. The User ID bar is where the name and/or patient number is specified. Simply click onto the white bar and type the patients name and/or number, when the photos or videos are taken they will be stored under that patients name and/or number in the file you have selected. The second white bar is the file destination bar. This is the destination that the images and video is being stored. The destination is generally a file, the desktop, or a folder. You can specify the file or folder that you would like to save these images in by clicking on the browse button directly to the right of the two white bars.

The last button on the screen is the PRINT Function. The **PRINT** button allows you to print directly to any networked computer in the office. This will allow you to print what is displayed on the screen to a networked printer that has been installed onto the computer.



170 East Kirkham Avenue

St. Louis, MO 63119

(800) 489-2282

(314) 968-2282

E-mail: [micro@seilerinst.com](mailto:micro@seilerinst.com)

[www.seilerinst.com](http://www.seilerinst.com)